

## PACKING GUIDELINES FOR “GOLD” SAMPLE TRANSPORT BOX.



1 Place the sample tubes into the absorbant pouch provided. All sample tubes must be contained in the pouches before further packing.



2 When the pouch is full, roll it up before inserting it into the green canister supplied.



3 Place the rolled absorbant pouch into a green canister. Each canister can accommodate three full pouches, a total of 27 samples.



4 Fill each canister in turn and place into the insulated box. Total capacity should be no more than 72 samples.



5 Once the canisters are in place insert one or two cold blocks into the box as required.



6 The cold blocks should be inserted as shown. Make sure they are seated firmly.






Next, switch on the small temperature data logger and insert into the box .



Finally, instal the insulated lid ensure it is sitting correctly in place.

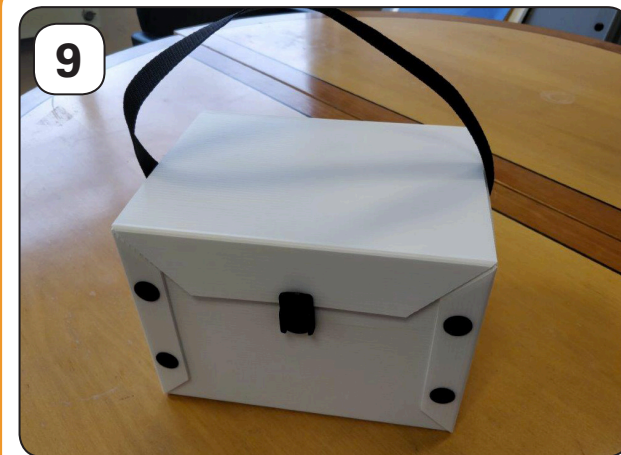

Secure the box lid in place. The samples are now ready for transporting.



Place any necessary paperwork in the document holder ready for transporting.



Pack two insulated boxes and all documents into the rucksack, secure with straps and close.



The rucksack and samples are now safe and ready to transport.